



HiPee Smart Posture Correction Wizard

Translated by
Google Translate
with assistance and editing by
Byronic @
www.strangetimez.com

IMPORTANT NOTE: There may be mistakes or errors in wording

User's Manual English

Certificate of Quality

Product Name: Smart Posture Correction Wizard

Model: P1

Inspector: Qualified

After Sales Unit: Tianjin Fruit Technology Co., Ltd.

Address: Haitai Green Industry Base, No. 6, Haitai Development Sixth Road,
Huayuan Industrial Zone, Tian
G-1101-15 to G-1101-32

Post Code: 300384

Phone: 400-900-5115

Storage

1. Storage Conditions:

Temperature: 10°C → 55°C

Relative humidity: < 80% RH non-condensing

Atmospheric Pressure: 70.0 ~ 106.0 kPa

2. The cleaned equipment must be thoroughly dried and stored in an environment meeting the requirements.

3. If you don't use it for a long time, please charge the device regularly to ensure battery life

4. During storage, protect the body of the device from falls and bumping continuously, to avoid damage.

5. Avoid direct sunlight, please store in a well-ventilated, non-corrosive gas environment, away from heating equipment and open flames.

Product Specifications

Name: Smart Pose Correction Wizard

Model: P1

Size: Large: 137x167x32mm Small: 120x132x32mm

Weight (including battery and collar): Large: 60 grams Small: 51 grams

Power Supply: Internal power supply

Rated input: 5V ~ 1A

Communications: Low energy Bluetooth 4.1 and above

CNIT ID: 2020DP11849

Executive Standard: GB4706.1-2005

Working Environment: -10°C → 55°C

Relative Humidity: < 80% RH non-condensing state

Atmospheric pressure: 70.0 ~ 106.0 kPa

Working Hours: 90 hours

Charge Time: Approx. 1.5 hours

Tianjin Fruit Technology Co., Ltd.

Address: Haitai Green Industry Base, No. 6, Haitai Development Sixth Road,
Huayuan Industrial Zone, Tianjin
G-1101-15 to G-1101-32

Company Website: www.fruittech.cn

Service Hotline: 400-900-5115

Postcode: 300384

Warranty Card

Name:

Contact Number:

Address:

Product Name:

Purchase Date:

Purchase Method:

Date	The cause of failure, and how it was handled

Warranty Description

1. From the date of purchase of this product, within the scope of normal use of the product according to the manual, if the product has performance failures that are not artificially damaged, the warranty period is 12 months. After 12 months, maintenance will be charged to the user.

2. Repairs that are not caused by the quality of our products, such as improper use, improper storage, unauthorized disassembly, etc., maintenance costs will be charged and the freight will be borne by the user.

3. When repairing the product, please send the warranty card and the purchased product to the after-sales service company.

Charging


When charging, please push the back of the device and remove it from the collar (as shown in the following (1) and (2)).

Charge Port: USB Type-C. The power adapter should meet these requirements:

Rated input: 5V ~ 1A.

Charging Time: approx. 1.5 Hours

Charging: **Red** light flashes Finished Charging: **Blue** light on

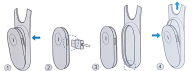

After charging is completed, please align the device with the groove of the neck ring when assembling, and pull the neck ring upward while installing (the pre-ring ??? is flexible and can be pulled but not too hard), and adjust the position of the device to ensure that its angle is perpendicular to the horizontal plane (see Figure 3 and 4 above).

Cleaning

Device Cleaning

1. The main body of this device is "not" waterproof, please do not wash it in water. If there is dust or stains, you can wipe the device with a clean dry cloth or a wrung wet cloth.
2. When cleaning, be sure to shut down the device and unplug the power adapter.
3. Do not use any detergent or cleaning agent for cleaning. If there is dirt that is difficult to clean, use an alcohol wipe. Wait for the alcohol to dissolve the dirt before wiping it down. The put it in a cool dark place and wait for the substance to evaporate completely, then install it in the collar.

Collar Cleaning

1. The collar can be removed and washed separately from the device. After washing and cleaning, please fully dry it in a cool dark place.
2. After disinfecting with an alcohol based disinfectant, please fully dry it in a cool dark place.
3. Once it is clean and dry, re-install the device.

Vibration and Indicator Light

Device Status	Vibration	Indicator Light
Boot Up	1 Short, 1 Long	Sufficient power: Blue light flashing Insufficient charge: Red light flashing
Shutdown	1 Long, 1 Short	*
Bluetooth Pairing	2 Short	*
Record Posture Success	2 Short	Blue light flashes fast, then flashes normally
Gesture Reminder	Default 3 Short, repeats	*
Sedentary Reminder	Default 3 Short, repeats	*
Charging On/Off	*	Charging: Red light flashing Finished Charging: Blue light stays on

Tips

1. This product is recommended for children, teenagers, and adults over 3 years old.
2. Before using this device for the first time, it is recommended to charge it fully
3. This device needs to be worn close to the body, wearing it through clothes may cause inaccurate data.
4. Avoid sharp objects, so the collar is not cut during use
5. Please do not modify or repair this device privately, if you need repair, please contact the after-sales service department

Instructions

1/ Boot Up

Press and hold the button for 3s to turn on the device (in the power-on state, long press the button for 3s, the device will shut down).



2/ Bind / Pair

Open WeChat and scan the QR code on the right, to follow the official account, and follow the prompts to bind the device.



3/ Use

Put on the device and follow the prompts in the applet to set the correct posture. After the posture is recorded successfully, you can customize the personalized settings in the applet / small program.



4/ Remind

When the posture is wrong or sitting for a long time, the device will vibrate to remind you. Click the button to turn on/off the reminder function.

